

Missions of Mercy

Protocol

Orientation for use of Chairside Suction Device

This must be read prior to use!

- Keep suction tip turned up when not in use!
- Turn suction on before turning tip down to place in patient's mouth.
- Keep suction 'on' when removing it from patient's mouth.
- Empty or notify appropriate person to empty waste after each patient.
- Run appropriate solution through suction prior to emptying.
- Thoroughly sanitize metal tip and lines after each patient.

Questions or concerns? See Dr. Dickinson or other appropriate staff for usage questions. 5/2015